



# **Confidence comes easy** with Eson 2

F&P Eson 2 was designed to meet the needs of patients and sleep professionals, at every important milestone in the CPAP therapy journey.

OVER 20 DESIGN IMPROVEMENTS COMPARED TO F&P ESON!





# Confidence comes easy... at every CPAP therapy milestone

To improve on a mask designed for performance and comfort, our development team focused on ease of use, providing you and your patient confidence at every step from titration through to adherence.

### **CPAP** therapy journey

#### FOR THE SLEEP PROFESSIONAL

#### **FOR THE PATIENT**



#### **SLEEP TEST**

## Selecting a nasal mask for titration

F&P Eson 2 retains the same RollFit™ Seal technology as F&P Eson, designed to deliver an effective seal to lessen the likelihood of leaks.

## FIRST PATIENT CONTACT

## Selecting a patient's mask for home use

The Intuitive headgear provides fitting options for patients, including the ability to stretch the mask directly over the head.

#### **SET UP**

## Fitting and demonstrating the mask

Blue highlights
- VisiBlue™ incorporated into
the mask provide
visual cues for
simpler, quicker and
more intuitive mask
education.

#### USE

## Patient begins using mask

A design focus on ease of fitting and ease of use lessens the likelihood of callbacks and improves patient confidence.


#### **ADHERENCE**

## Long-term adherence

Eson 2 has been designed to not just work on the first night, but to increase the chance of long-term therapy success.



## **F&P Eson**<sup>™</sup>2 **Key features and benefits**

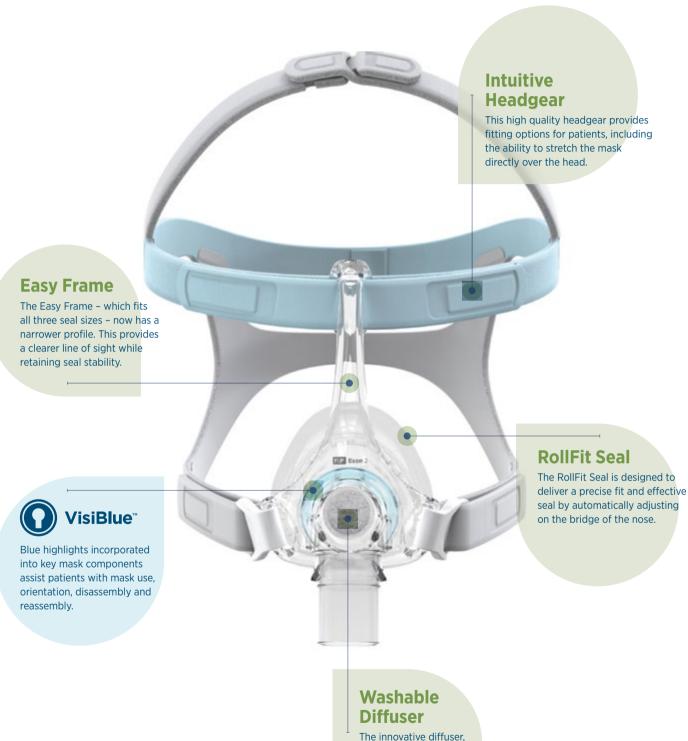

which reduces noise and draft, is now washable and more durable.

### Fitting the Eson 2



Hold open the bottom straps of the Headgear with the Mask Frame facing downwards.



Guide the headgear over your head so that the silicone seal is resting on your forehead.



Place your hands on the bottom straps on either side of the mask frame and pull out and downwards. Ensure the silicone seal rests over your nose and the blue top strap is sitting on your forehead.



Gently tighten the headgear straps of the mask headgear starting with the blue forehead straps followed by the bottom straps.

MASKS		400ESN221	ESON2 Headgear Small
ESN2SA	ESON2 Small	400ESN222	ESON2 Headgear Medium-Large
ESN2MA	ESON2 Medium	400ESN231	ESON2 Mask No Headgear Small
ESN2LA	ESON2 Large	400ESN232	ESON2 Mask No Headgear Medium
		400ESN233	ESON2 Mask No Headgear Large
SPARE PARTS		400FPH241	F&P Swivel 2
400ESN214	ESON2 Seal Small	400ESN251	ESON2 Clips
400ESN215	ESON2 Seal Medium	400ESN261	ESON2 Diffuser

